

Maxtec Technical Note

- **Cleaning and Disinfecting the Exhalometer™**

For cleaning of the Exhalometer™ the recommended disinfectant is alcohol based. We selected an alcohol based material because it is fast drying. The presence of liquid in the flow tube of the Exhalometer™ could alter its accuracy, therefore we recommend waiting for a minimum of 10 minutes after cleaning for the equipment to fully dry.

Any recognized alcohol based disinfectant should be suitable.

- **Manufacturer's Cleaning Recommendations:**

1. After use, clean and disinfect the flow tube with one spray of ALPET D2* sanitizer into each end of the flow tube. Allow to dry before using.
2. Clean and disinfect the outside surfaces by spraying with ALPET D2*. Allow surfaces to fully dry.

***Available from:**

Bestsanitizers Inc.

P.O. Box 1360

Penn Valley, CA 95946

888-225-3267

www.bestsanitizers.com